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Producttype + artienummer:

ROALINO – Premis Allround Indoor Outdoor Wheeled walker with pushbar, tray and basket.

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Product description:

Design lightweight wheeled walker for indoor and outdoor use, with foam covered push bar, bar brakes, plastic tray and fabric basket underneath. 8" wheels, aluminum frame. The walker is foldable and available in the frame colours titanium gray and titanium silver, both with a black tray and basket.

Is this product the right fit for you?

The product you have bought has a number of specifications one must keep in mind. These specifications are described underneath, so that you can check if this product suits your needs.

Product dimensions (l*w*h):	Handle height 82 – 92 cm Length overall 52 cm Width overall 72 cm Width when folded 31 cm
Product weight:	7.4 kg
Maximum user weight:	136 kg
Wheel diameter:	8 inch – 20.32 cm
Other specifications:	Nylon basket (40*30*22 cm) Tray (42*30*3 cm) Pushbar with bar brakes

If any of these specifications is not suitable for your needs, please contact the retailer as soon as possible!

Version:	1.0	Classification:	Public use
Status:	In use		
Date:	27-05-2020	Author:	R.Keuning

User Manual

Premis Allround Indoor Outdoor Wheeled Walker

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Warnings and Safety Instructions:

15. The maximum acceptable weight of the user is 136 kg and may not be exceeded.
16. Check the brakes before use.
17. All wheels must touch the floor at all times.
18. Do not use the walker on escalators, stairs, moving sidewalks or uneven surfaces.
19. The tray is not suitable for cleaning in dishwashers or hot pans.
20. Pay attention to your fingers when using the folding mechanism. Keep the user instructions in mind.
21. Only lift the walker if you are physically capable to do so and do not lift the walker above your power. Lift the walker with your legs and not with your back.
22. Only lift the walker by holding the frame. The walker is not meant to be lifted by the handles, bars, wheels or parts of the basket and tray suspension.
23. To prevent damage and turn over: do not hang items on the walker that are not meant for this. Only use items that are specifically designed or approved by Premis for this purpose.
24. Do not make any changes to the exterior of the walker without written permission of Premis.
25. Only repair or change parts of the walker with original parts.
26. Make sure the walker is maintained and checked at least once a year.
27. Make sure that you walk in between the hind wheels when using the walker and maintain as much of an upright posture as possible, to prevent falling.
28. Do not use the walker if it's damaged or incomplete.

Included in the packaging of this product:

- 1x Wheeled walker ROALINO with push bar and bar brakes
- 1x Tray (plastic)
- 1x Basket (Nylon fabric) (attached to walker)

Intended use of the product:

The Premis Alino walker with pushbar and bar brakes is intended for use by people with reduced mobility and are in need of extra support when walking and standing. The walker offers support and stability, so that people can remain independent as long as possible, both indoors and outdoors. The pushbar is suitable for people with reduced hand functions; the walker can be steered and braked with one hand.

Version:	1.0	Classification:	Public use
Status:	In use		
Date:	27-05-2020	Author:	R.Keuning

User Manual

Premis Allround Indoor Outdoor Wheeled Walker

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Contra-indications of the product:

The walker is not suitable for people who can only walk short distances. The walker has no seat and therefore does not provide the possibility to sit down and rest whenever needed.

The walker is not suitable for Parkinson patients or patients with dementia, who can not control the walker on their own at all times.

The walker is not suitable for people with reduced eye sight.

The walker is not suitable for independent use by people with mental disabilities.

The walker is not suitable for people who cannot stand upright independently.

Clinical benefits:

This walker provides more and longer independency to people that have reduced mobility. The walker supports walking and standing, and therefore reduces the risk of falling. It also costs people less energy to stand and walk, so that they can keep it up for longer amounts of time and walk longer distances.

The pushbar with bar brakes also provides people that have both reduced mobility and reduced hand functions, because the walker can be operated with just one hand.

Install before use:

7. Place the pushbar into the frame (Picture 1). Place the walker upright.
8. Fold down the already installed basket (Picture 2).
9. Unfold the walker by pushing the frontwheel support and hindwheel support apart. (Picture 3).
10. Place the plastic folding fixture on the center of the frame, to ensure the walker will not suddenly fold back in when in use. (Picture 4).
11. Place the tray in the metal tray suspension (Picture 5).
12. Place the cable guard on the frame. (Picture 6).

Before the first use:

3. Adjust the height of the pushbar to the correct height. You can measure the correct height by measuring the distance between the floor and your wristjoint, when you have your arms hanging down alongside your body.
4. Press the plastic fixture to fixate the pushbar on the correct height. Make sure that the height of the bar is equal on both the left and right side. (Picture 7).

Version:	1.0	Classification:	Public use
Status:	In use		
Date:	27-05-2020	Author:	R.Keuning

User Manual

Premis Allround Indoor Outdoor Wheeled Walker

Version:	1.0	Classification:	Public use
Status:	In use		
Date:	27-05-2020	Author:	R.Keuning

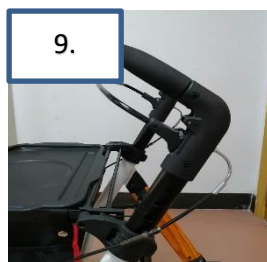
User Manual

Premis Allround Indoor Outdoor Wheeled Walker

User Instructions:

Locking and unlocking the brakes

3. Push the bar brakes down until you can hear a clicking sound. The brakes are now locked. (Picture 8).
4. Press the bar brakes to the pushbar as far as possible to unlock the brakes. (Picture 9).

Braking

2. Press the bar brakes to brake. The further you press the bar brakes, the more the brakes slow down the walker.

Transport or storing the walker

4. Remove the plastic tray. (Picture 10).
5. Remove the folding fixture from the middle frame and fold down the basket. (Picture 11).
6. Push the front and back frame towards each other to fold in the walker. The walker can stand upright on its own when folded. (Picture 12).


Version:	1.0	Classification:	Public use
Status:	In use		
Date:	27-05-2020	Author:	R.Keuning

User Manual

Premis Allround Indoor Outdoor Wheeled Walker

Adjusting the brake intensity

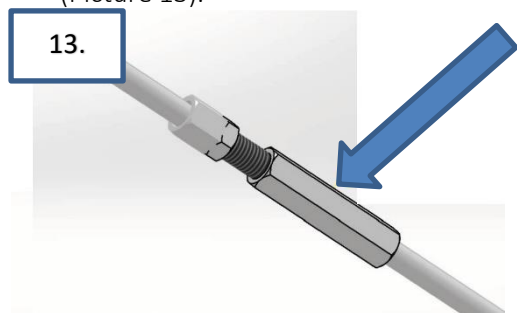- Turn the brake adjustment clockwise to tighten the brakes and improve brake intensity. (Picture 13).
- Turn the brake adjustment counter-clockwise to loosen the brakes and decrease brake intensity. (Picture 13).

Cleaning of the product:

- Clean the walker regularly with a damp towel and a non-aggressive soap. Do not use acidic or scrubbing cleaning detergents. Test the cleaning product on an unnoticeable spot on the walker before cleaning to make sure it does not damage the walker. Do not apply any cleaning product directly onto the walker.
- Clean the tray by hand. Do NOT put the tray in a dishwasher!

Maintenance of the product:

- When the walker is not in use; store the walker in a covered and dry area, to prevent oxidation and rust.
- Clean the walker regularly to remove and prevent collection of dirt on moving parts.
- Ensure that the walker is maintained and checked at least once a year to replace worn parts before they break. Not maintaining the walker is a direct void of any warranty and may lead to decreased safety when using the product.

Version:	1.0	Classification:	Public use
Status:	In use		
Date:	27-05-2020	Author:	R.Keuning

User Manual

Premis Allround Indoor Outdoor Wheeled Walker

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Technical specifications:

Used materials:	Aluminum, polyurethane, galvanised steel, varnish
Product dimensions:	Handle height: 82 – 92 cm Length: 52 cm Width: 72 cm Width when folded: 31 cm Wheel diameter: 8 inch – 20,32 cm
Product weight:	7.4 kg
Maximal user weight:	136 kg
Periodically replaceable parts:	Wheel bearings PU wheels Brake cables Brake blocks

Warranty:

The warranty period starts on the date of purchase. When ordered online this date is the shipping date as provided on the invoice.

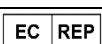
3. Void of warranty:

- a. Using the product for any purpose or in any manner other than its intended use as described in this manual.
- b. When a change or repair of the product is made, which has not been authorized by Premis Medical.

4. Warranty definitions:

- a. Shipping fees for returning a product are not included in the warranty. A return form must be requested and filled out before shipping the product to Premis Medical.
- b. The warranty term of the product amounts to 24 months starting from the date of purchase.
- c. The warranty term of wear sensitive parts and dynamic parts of the product amounts to 6 months starting from the date of purchase.

Explanation of symbols:

Symbol	Definition	Symbol	Definition
			Manufacturer
	Production date		Approved by the European Union
SN	Serial Number		Authorized European Representative
	User Manual is available online		Read the User Manual before use

Version:	1.0	Classification:	Public use
Status:	In use		
Date:	27-05-2020	Author:	R.Keuning

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Troubleshooting:

Potential problems	Potential causes	Potential solutions
The wheels do not roll anymore.	The wheel bearings are worn out.	Replacet he wheel bearings.
The handles turn or come loose.	The handles are worn out or stretched.	Replace the handles or glue them down with contact glue.
One of the wheels is floating / does not touch the floor.	3. The walker is not properly unfolded OR 4. The frame is crooked	3. Unfold the walker completely OR 4. Get the frame repaired
Folding the new walker is difficult	The walker is not yet properly walked in.	Fold and unfold the walker several times in a row until it goes smoother.
My hands/arms/shoulders start to hurt when I use the walker.	3. The pushbar is not properly OR 4. You experience a lot of vibrations from the uneven surface you walk on.	3. Correctly adjust the pushbar OR 4. Replace the PU Wheels OR Avoid walking on rough or uneven surfaces
The brakes do not brake sufficiently.	3. The brake blocks are worn out OR 4. The brakes are not adjusted correctly.	3. Replace the brake blocks OR 4. Adjsut the brake intensity
The brake cables get caught on door handles and the like.	The brake cable guides have come off.	Replace the brake cable guides.

What to do in case of a severe accident?

In case a serious accident happens that involves this walking aid, we request you to inform the manufacturer as soon as possible. If necessary the manufacturer will be able to take measures to prevent future accidents and guarantee a safe use of the product.

Version:	1.0	Classification:	Public use
Status:	In use		
Date:	27-05-2020	Author:	R.Keuning

User Manual


Premis Allround Indoor Outdoor Wheeled Walker

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Guidelines and regulations

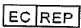- All products sold by Premis Medical are CE approved. Furthermore, we meet the international ISO standards for Medical Aids through our ISO 9001 certification. Therefore, you can trust on a safe use of the product you purchased and are guaranteed of a quality product.
- When applicable our products meet the MDD and MDR regulations.
- All products of Premis Medical meet the applicable laws and regulations.

 Manufacturer
Premis Medical BV. De nort 20, 3931 NG, Woudenberg The Netherlands
 

Version:	1.0	Classification:	Public use
Status:	In use		
Date:	27-05-2020	Author:	R.Keuning