YOU-Q

LUCA XL

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© 2016 Sunrise Medical

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorisation of Sunrise Medical.

The information provided is based on general data concerning the constructions known at the time of the publication of this manual. Sunrise Medical executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. Sunrise Medical cannot be held liable for possible damage resulting from specifications of the product deviating from the standard configuration.

The available information has been prepared with all possible diligence, but Sunrise Medical cannot be held liable for possible errors in the information or the consequences thereof. Sunrise Medical accepts no liability for loss resulting from work executed by third parties.

The management system of SUNRISE MEDICAL is certified to ISO 9001, ISO 13485 and ISO 14001.

2016-01

1	Foreword	34
1.1	This user manual	34
1.2	Symbols used in this manual	34
2	Safety	35
2.1	Temperature	35
2.2	Moving parts	35
2.3	Electromagnetic radiation.....	35
2.4	Decals and instructions on the wheelchair.....	36
3	General description of the wheelchair	37
3.1	Configurations.....	37
3.2	The user.....	38
3.3	User environment.....	38
3.4	Adjustment options	39
4	Wheelchair settings	40
4.1	Seat angle, seat height	40
4.2	Seat position and centre point of gravity setting.....	40
4.3	Height and depth of the controller	40
4.4	Programme of the control system	40
5	Checking the wheelchair before putting into use	41
6	Use of the wheelchair	42
6.1	Getting in and out of the wheelchair.....	42
6.2	Driving the wheelchair.....	43
6.3	Driving on a slope	43
6.4	Obstakels.....	44
6.5	Adjustment options	45
6.6	Freewheel switch: pushing the wheelchair.....	45
6.7	Storage after use	46
7	Transport of the wheelchair.....	47
7.1	Transporting the wheelchair.....	47
7.2	Transport of the wheelchair (no occupant).....	47
7.3	The wheelchair as seat in a vehicle	48
7.4	Securing the occupant	48
8	Maintenance of the wheelchair	50
8.1	Cleaning the wheelchair.....	50
8.2	Batteries.....	51
8.3	Tyres.....	52
8.4	Discarding the wheelchair.....	52
9	Warranty	53
9.1	Definitions of terms	53
9.2	Warranty period table.....	54
10	Technical specifications	56
10.1	Electric wiring diagram.....	56
10.2	Technical specifications Luca XL	57
10.3	Technical specifications XL seating system.....	57

1 Foreword

1.1 This user manual

The user manual for this electric wheelchair consists of three booklets. Read the entire user manual carefully before taking the product into use. The information provided by this manual is essential for the safe use and proper care (cleaning) of the wheelchair.

- The general user manual (this booklet)
- The user manual for the seating system (XL Seating)
- The user manual for the controller

If one of these user manuals was not included with your wheelchair, please contact your dealer immediately. In addition to this user manual, there is also a service manual for qualified specialists.

This general user manual will refer, where necessary, to one of the other user manuals. This is indicated as follows:

 **GENERAL**

 **CONTROLLER:** Refers to the user manual for the controllers.

 **XL SEATING:** Refers to the user manual for the seating system.

1.2 For further information

Please contact your local, authorised Sunrise Medical dealer if you have any questions regarding the use, maintenance or safety of your wheelchair. In case there is no authorised dealer in your area or you have any questions, contact Sunrise Medical either in writing or by telephone.

For information about product safety notices and product recalls, go to www.SunriseMedical.eu

1.3 Symbols used in this manual

Warning symbol

Follow the instructions next to this symbol closely.

Not paying careful attention to these instructions could result in physical injury or damage to the wheelchair or the environment.

Reference symbol

The symbol refers to a separate user manual. This reference will indicate the specific user manual and the section to which is being referred.

2 Safety

Follow the instructions carefully next to these warning symbols! Not paying careful attention to these instructions could result in physical injury or damage to the wheelchair or the environment. Wherever possible, safety information is provided in the relevant chapter.

Safety information is indicated with the warning symbol.

2.1 Temperature

- **Avoid physical contact with the wheelchair's motors at all times. Motors are continuously in motion during use and can reach high temperatures. After use, the motors will cool down slowly. Physical contact could cause burns.**
- **If you do not use the wheelchair, ensure that it is not exposed to direct sunlight for lengthy periods of time. Certain parts of the wheelchair, such as the seat, the back and the armrests can become hot if they have been exposed to full sunlight for too long. This may cause burns or allergic reactions to the skin.**

2.2 Moving parts

A wheelchair has moving and rotating parts. Contact with moving parts may result in serious physical injury or damage to the wheelchair. Contact with the moving parts of the wheelchair should be avoided.

- **Wheels (turning and castor)**
- **Electric tilt in space adjustment**
- **Electric backrest adjustment**
- **Swing-away controller holder**

2.3 Electromagnetic radiation

The standard version of your electric wheelchair has been tested on the applicable requirements with respect to electromagnetic radiation (EMC requirements) In spite of these tests:

- it cannot be excluded that electromagnetic radiation may have an influence on the wheelchair. For example:
 - mobile telephony
 - large-scale medical apparatus
 - other sources of electromagnetic radiation
- it cannot be excluded that the wheelchair may interfere with electromagnetic fields. For example:
 - shop doors
 - burglar alarm systems in shops
 - garage door openers

In the unlikely event that such problems do occur, we request that you notify your dealer immediately.

2.4 Decals and instructions on the wheelchair

The signs, symbols and instructions affixed to the wheelchair comprise part of the safety facilities. They must never be covered or removed. They must remain present and clearly legible throughout the entire lifespan of the wheelchair.

Replace or repair all illegible or damaged signs, symbols and instructions immediately. Please contact your dealer for assistance.

See figure 2.1

1. Check manual before using.
2. Battery charging connection.
3. Danger for crushing!

Use caution when swinging the controller aside to avoid getting anything crushed.

4. Attachment point of the tie-down system for transportation in a vehicle.
5. Freewheel switch

Don't put the freewheel switch in 'Push' mode on a slope

6. Trap danger. Stay away from moving or rotating parts with hands and fingers when adjusting your seating options.
7. Nameplate (Fig.7.2)
Please refer to the table at the end of the manual

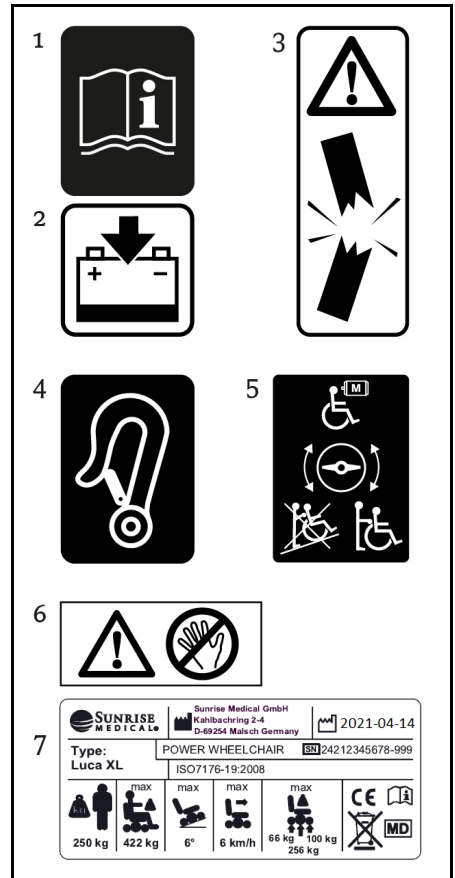
Figure 2.1

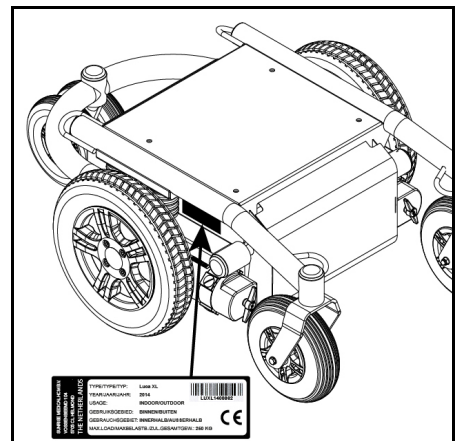
Figure 2.2

3 General description of the wheelchair

3.1 Configurations

Generally speaking, the Luca XL is comprised of an undercarriage with a seating system on top. The Luca XL is configured as a midwheel-driven wheelchair (MWD). The max. speed is 6 km/h.

- **The technical specifications may not be changed.**
- **Do not make any changes to the electrical circuit.**
- **Any modifications to the wheelchair or any parts thereof are not permitted.**

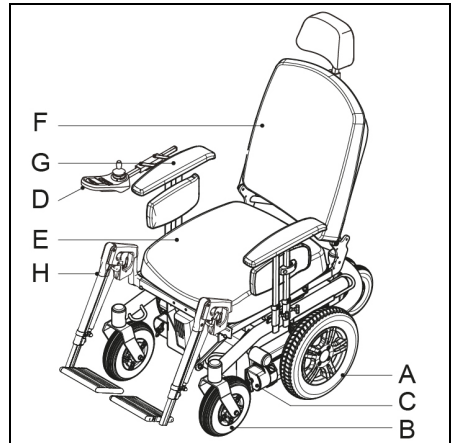
Figure 3.1

Main components basic model

- a. Drive wheels
- b. Castors wheels
- c. Freewheel switch
- d. Controller

Zitsysteem

- e. Seat cushion
- f. Backrest
- g. Armrest
- h. Legrest with footsupport

3.2 The user

Driving an electric wheelchair requires cognitive, physical and visual skills. The user must be able to estimate and correct the results of actions when operating the wheelchair.

The wheelchair can not transport more than 1 person at a time. The maximum user weight is 250 kg. unless otherwise indicated on the identification plate. Additional weight, such as rucksacks, accessories or medical apparatus must be added to the user's weight to determine the maximum weight, which cannot be exceeded.

The user must be informed of the contents of this user manual before driving the wheelchair. In addition, the user of the wheelchair must be given thorough instruction by a qualified specialist before he or she participates in traffic. The first sessions in the wheelchair should be practiced under supervision of a trainer/advisor.

- **The user of the wheelchair is at all times completely responsible for complying with the applicable local safety regulations and guidelines.**
- **If you are under the influence of medicines that can have an effect on your ability to drive, you are not permitted to drive a wheelchair.**
- **Adequate vision is required in order to safely operate a wheelchair in the user situation concerned.**
- **Not more than one person at a time can be seated in the wheelchair.**
- **Do not allow children to ride in the wheelchair unsupervised.**

3.3 User environment

This wheelchair has been designed to accommodate both indoor and outdoor use (EN12184 (2009) class B). When driving the wheelchair outdoors, drive only on paved roads, pavements, footpaths and bicycle paths. The speed must be adapted to suit the environment. With all adjustment options set to neutral (table 3.1), it can be assumed that when using a Luca XL, a slope of $\leq 6^\circ$ belongs to a normal user environment with no danger of instability. In the technical specifications, you will find this under 'maximum safe slope'.


- **Drive carefully on slippery roads resulting from rain, ice or snow!**
- **Prevent the wheelchair from coming into contact with sea water: sea water is caustic and may damage the wheelchair.**
- **Prevent the wheelchair from coming into contact with sand: sand can permeate into the moving parts of the wheelchair, causing extensive wear on these parts.**
- **You are required to turn on the lights in case of limited visibility.**
- **When driving at higher speeds you must be extra careful. Select a lower maximum speed indoors, on the pavement and in pedestrian areas.**
- **Do not drive off high obstacles.**
- **Do not use the wheelchair if temperatures are below: -10°C or above $+50^\circ\text{C}$.**
- **Do not attach a weight to the wheelchair without the approval of a qualified specialist. This may negatively affect the stability of the product.**
- **Do not push and/or tow any objects with the wheelchair.**
- **Do not open doors using the legrests.**
- **Do not drive through puddles of water.**

3.4 Adjustment options

The Luca XL is available with the following electrical and/or mechanical adjustment options:

Adjustment	Neutral position
Powered tilt in space	The entire chair upright
Powered backrest recline	The backrest as upright as possible
Powered (or mechanical) elevating legrests	The legrests are positioned as closely as possible to the chair.

Table 3.1: Powered options and their neutral position.

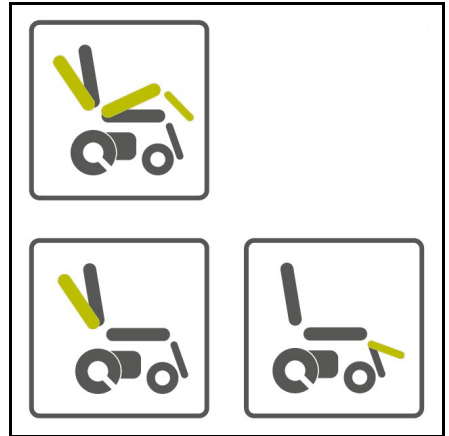Figuur 3.2

- Use of the electrical adjustment options may influence the centre point of gravity. These adjustments should only be made when the wheelchair is stationed on a flat surface.
- Electrical adjustment options are linked to moving and/or rotating parts. Contact with these moving parts may result in serious physical injury or damage to the wheelchair. Contact with the moving parts of the wheelchair must be avoided.

4 Wheelchair settings

The average wheelchair user does not exist. This is why Sunrise Medical wheelchairs can be adjusted according to the specific needs of the user. In this, we make a distinction between wheelchair settings and wheelchair adjustment options. Settings are carried out only once and must be executed by qualified specialists, unless explicitly stated otherwise. Adjustment options can be carried out by the user. These adjustment options do not require the use of tools.

Only to be carried out by qualified specialists.

Adjusting the seating system

The extensive settings with respect to the seating system allow for optimum support of the body. The following setting options are possible:

Seating system **XL SEATING**

- Depth, width of the seat
- Backrest angle
- Armrest height
- Lower leg length
- Angle of the footplate

4.1 Seat angle, seat height

Depending on the specific circumstances of the user of the wheelchair the seat height can be set. Every optional seat height lays within the 6° (10.5%) maximum safe slope of according to EN12184 (2009) ClassB. Lowering the seat height has a positive influence on the (dynamic) stability.

Enlarging the seat height has a negative influence on the (dynamic) stability, because the centre point of gravity gets higher.

4.2 Seat position and centre point of gravity setting

The seat position and centre point of gravity can be adjusted. For this purpose there are 3 positions in the seat slide. The wheelchair is designed and tested to be used in the standard position (middle hole). Only change the seat position and centre point of gravity to compensate specific user circumstances. For example: when a user has no legs.

Changing the seat position and centre point of gravity:

- Will have a negative influence on the driving characteristics of the wheelchair if used wrong.
- Will have a negative influence on the (dynamic) stability of the wheelchair if used wrong.

In case of doubt contact Sunrise Medical technical support for advice.

4.3 Height and depth of the controller

See  **XL SEATING**

4.4 Programme of the control system

Only to be carried out by qualified specialists. Incorrect setting of the parameters of the control system could lead to very dangerous situations.

5 Checking the wheelchair before putting into use

Be careful! Check the following before driving:

- Are the tyres sufficiently inflated (Ch. 8.3).
- Are the batteries sufficiently charged? The green lights on the battery indicator must be on. 
CONTROLLER
- Are the lights on the direction indicators functioning properly? 
CONTROLLER
- Ensure that the free wheel switch has been set to 'drive'.

When operating the wheelchair, ensure that your clothing does not hamper the wheelchair (i.e. too long). Before use, always check if your clothing or accessories do not come into contact with the wheels or and other moving and/or rotating parts in which they could become entangled.

In winter, batteries have a reduced capacity. During a period of light frost, the capacity is roughly 75% of the normal capacity. At temperatures below -5°C this will be roughly 50%. This will reduce your range of action.

6 Use of the wheelchair

6.1 Getting in and out of the wheelchair

Getting in and out of a wheelchair is sometimes called making a transfer.

Be careful! Before a transfer is made, ensure that:

- The controller has been switched off **CONTROLLER**
- That the freewheel switch has been switched to 'drive' (ch. 6.6)
- Do not stand on the foot-plates. These have not been designed to accommodate the full weight of a person. In addition, this could cause the wheelchair to topple over.

Forward transfer

Flip the footplates up (fig 6.1).

Swing the calf plates to the side (if applicable) (fig 6.1).

If possible, swing the legrests to the side (fig 6.2).

Seat yourself in the chair.

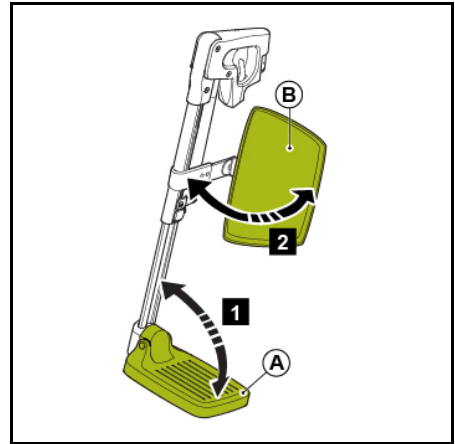
Figure 6.1

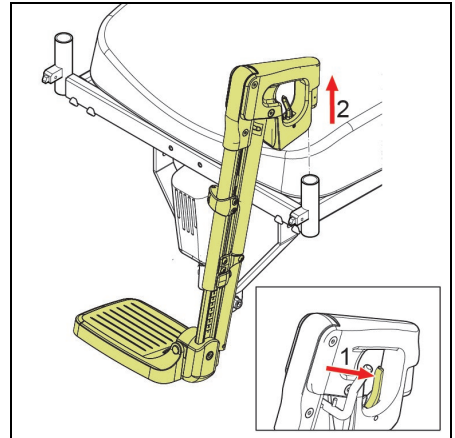
Figure 6.2

Sideways transfer

Push the lever to the back to unlock the armrest (fig 6.3).

Take the armrest out (fig 6.3).

Seat yourself in the chair.

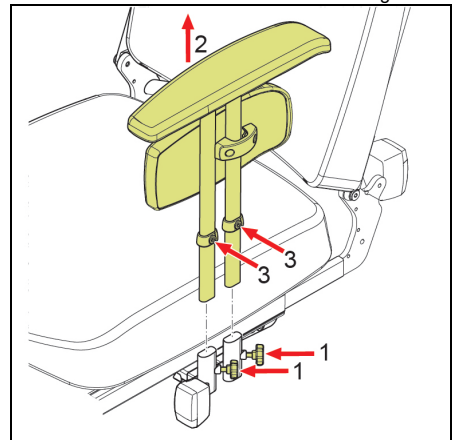
Figure 6.3

6.2 Driving the wheelchair

- As a wheelchair user, you are vulnerable in traffic. Keep in mind that other participants in traffic may not always notice you. Keep to the applicable traffic regulations.
- Avoid out-of-the-way routes so that help can be quickly on its way when needed.
- Adapt your driving style and speed to the circumstances.

Electric wheelchairs are driven by means of a controller. The controller has its own user manual  **CONTROLLER**, which has been included with this wheelchair.

- Switch on the controller  **CONTROLLER**
- Set the maximum speed limit  **CONTROLLER**
- Move the joystick into the direction you would like to go
- Moving the joystick even further forwards will cause the wheelchair to move faster

Curves

- Never take curves at full speed. Decrease your speed before taking a curve.
- Use the indicators when changing direction.

Braking/Emergency stop

To brake, the joystick must be moved into the neutral position or released completely. The wheelchair will brake automatically. An emergency stop can also be made by releasing the joystick.

An emergency stop can be made by releasing the joystick. The following should be avoided when driving:

- Do not push the on/off button
- Do not move suddenly in the opposite direction. The wheelchair will come to an abrupt standstill with a sudden jolt. Not only is this uncomfortable, it could lead to tipping over if you are on a slope.

6.3 Driving on a slope

With all adjustment options set to the neutral position (table 3.1), it can be assumed that when using a Luca XL, a grade of $\leq 6^\circ$ belongs to a normal user environment with no danger of instability. In the technical specifications, you will find this under 'maximum safe slope'.

Slopes steeper than the normal user environment

Driving on slopes whose grade is steeper than the maximum safety limit can lead to safety risks with respect to stability. This requires great care and complete control of the user. Always drive carefully and never take unnecessary risks! Follow the instructions next to the warnings carefully!

The stability and performance of a wheelchair depend on a number of variables. Wheelchairs are adapted according to the needs of individual users. This is why variables will differ from wheelchair to wheelchair. Ask your dealer to inform you about the instructions for use and the specific settings and adjustments that can influence the driving characteristics of the wheelchair.

Driving on a slope

- When driving on a slope, always drive carefully and with the utmost concentration.
- Avoid sudden and jerky movements.
- Avoid making emergency stops on a slope.
- Avoid changing direction on a slope whenever possible.
- Do not turn on a slope.
- Only drive up a slope with all the adjustment options set to the neutral position.
- Driving on a slope in reverse can be extremely dangerous.
- Do not drive on slopes with loose gravel or a sandy surface, as one of the driving wheels could slip/spin.
- Driving on a slope too long can cause overheating of the motor.

Uphill

- If you notice a strong decrease in speed when driving up a slope, take a less steep route.

Downhill

- Prevent the wheelchair from gaining too much speed.

6.4 Obstacles

Driving up a kerb

- Choose the place where the kerb is at its lowest (see chapter 10 'climbing capacity for obstacles')
- Drive straight up to the kerb (20 cm in front of the kerb).
- Move the joystick forwards. Drive up the kerb without changing direction.
- As soon as the front wheels are on the pavement, you must maintain speed in order to get onto the pavement with all wheels. If it is impossible to drive onto the pavement, find a lower place where it is possible.

Driving down a kerb

- Choose the place where the kerb is at its lowest. In case of doubt, do not take any risks and find another route or someone to help you.
- Drive with your front wheels straight up to the kerb.
- Move the joystick slowly forwards. Drive down the kerb carefully and as slowly as possible, without changing direction.

- Never drive down stairs in a wheelchair.
- Do not drive off obstacles higher than 6 cm.

6.5 Adjustment options

Adjustment options are those parts of a wheelchair that can be adjusted by the user without the use of tools. The Luca XL can be ordered with the following adjustment options:

Mechanical adjustment options

- Comfort legrests  **XL SEATING**
- Swing-away joystick holder  **XL SEATING**
- Headrest  **XL SEATING**

Electrical adjustment options

- Electrical tilt adjustment  **XL SEATING**
- Electrical high/low option  **XL SEATING**
- Electrical backrest adjustment  **CONTROLLER**
- Electrical legrests  **CONTROLLER**

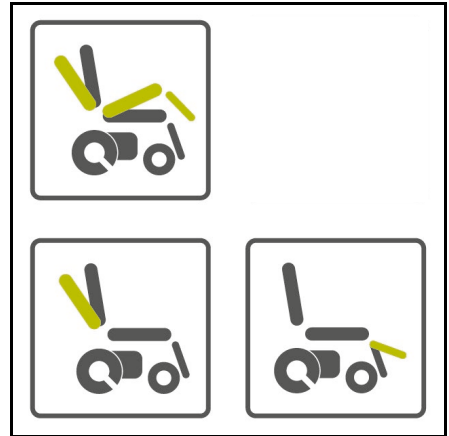
Figure 6.4

6.6 Freewheel switch: pushing the wheelchair

The wheelchair can also be moved by pushing. For this purpose, the motors must be disengaged. This is done with a freewheel switch; one for each motor.

The freewheel switch can be set to two positions:

- 'Drive' mode. In this position, the chair can not be pushed (a in figure 6.5).
- 'Push' mode. In this position, the chair can not be driven using the controller (b in figure 6.5).

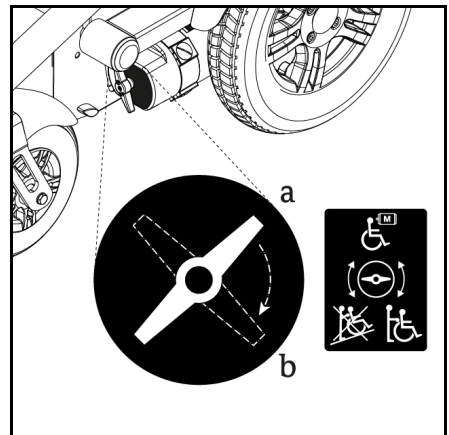
Figure 6.5

The automatic parking brake only works if the switch is set to the 'drive' position. The freewheel switch should only be set to 'push' if the chair is to be pushed. When the chair is no longer being pushed, the freewheel switch should be set to 'drive' immediately.

- **The freewheel switch is a function that was developed especially for wheelchair attendants. The freewheel switch should only be operated by the attendant and never by the user.**
- **Never set the switch to 'push' on a slope! When the freewheel switch is set to 'push', the automatic parking brake will be deactivated. The wheelchair could then move down the slope.**

6.7 Storage after use

If the wheelchair is not in use, it must be stored in a cool place without being exposed to extreme weather conditions. Do not place the wheelchair in direct sunlight. Parts of the wheelchair can become so hot that they could cause burns. The surrounding temperature when stored may not be lower than 20°C or higher than +65°C.

7 Transport of the wheelchair

7.1 Transporting the wheelchair

Power wheelchairs are very heavy to lift. Use suitable ramps to wheel the chair in and out of the vehicle. Once the wheelchair is in the vehicle, it must be secured with an ISO 10542 approved tie-down system that is suited to the weight of that particular wheelchair. The wheelchair can only be fastened by securing it to the tie-down points on the wheelchair's frame (fig. 7.1). The total weight of the wheelchair depends on its composition. The technical specifications (Ch. 10) give insight in several optional items and how they influence the total weight. Always weigh the total weight of the wheelchair to be certain that the correct fastening system is used.

- Position the wheelchair in the car never in another way than facing front wards.
- Use an ISO 10542 approved attachment system suitable for the total weight of the wheelchair to secure the wheelchair.
- The wheelchair can only be fastened by securing it to the tie-down points on the wheelchair's frame (fig. 7.1). The wheelchair may not be secured onto any accessories (wishbones, armrests, anti-tip brackets etc.).
- Use all 4 tie-down points (fig. 7.1).
- Make sure that the freewheel levers are set to 'drive' (fig. 6.5).
- It's not allowed to alter the tie-down points on the frame without permission of Sunrise Medical.
- If the wheelchair is involved in an accident it needs to be checked and approved by a Sunrise Medical expert before it will be used again.
- Only use GEL or AGM batteries.

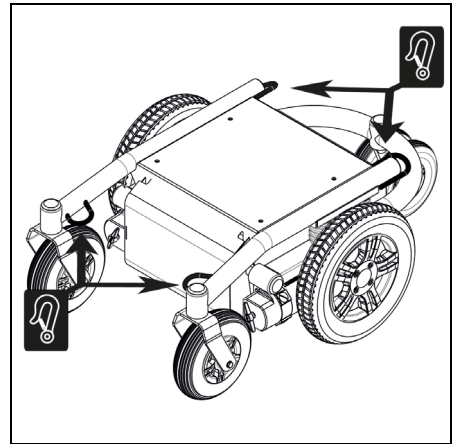
Figure 7.1

7.2 Transport of the wheelchair (no occupant)

When the wheelchair is transported without an occupant, remove the parts that can be taken away.

- Remove the legrests  **XL SEATING**
- Remove the armrests  **XL SEATING**
- Remove the seating  **XL SEATING**
- Remove the backrest  **XL SEATING**
- Remove the headrest (if applicable)  **XL SEATING**
- Store and secure these items in a proper way. It's also possible to flip down the backrest if that is necessary due to lack of space in the vehicle  **XL SEATING**.

7.3 The wheelchair as seat in a vehicle

The Luca XL is successfully crash tested according to ISO 7176-19. Even so, a wheelchair is not designed as a car seat and cannot offer the same degree of safety that is offered by standard car seats. Sunrise Medical recommends that wheelchair users transfer to a regular car seat if possible.

It's not possible for every wheelchair user to make a transfer. To optimize the safety of the wheelchair user and other passengers in the vehicle. Follow all the instructions and warnings carefully!

- Always ask for confirmation of the transporter that the vehicle is suitably designed, insured and equipped to transport a person in a wheelchair.

Fixating the wheelchair

- To tie down the wheelchair for transport in the vehicle, follow the instructions of chapter 7.1.

- When the wheelchair is equipped with a table top this needs to be removed during transport. Store these parts securely.
- All adjustment options of the seating system need to be in the 'neutral' position (Ch. 3.4).
- Sunrise Medical recommends that the wheelchair is always equipped with a headrest and that this is used during transport when the wheelchair is used as a carseat.
- Any accessories on the wheelchair need to be removed and need to be stored securely..

7.4 Securing the occupant

- Use a three-point occupant restraint system to secure the occupant of the wheelchair on the pelvis/hips and shoulder. This three-point safety belt must be secured to the roof and the floor of the vehicle (fig.7.2).
- Position the belt across the hips as tightly as possible at an angle between 30° and 75°.
- The other part of the belt is positioned to sit across the chest and shoulder (fig. 7.2).
- The seat belt should fit as tightly as possible and must not be twisted.
- Ensure that the seat belt is not obstructed from having contact with the body by wheelchair parts, such as armrests or wheels.
- When the wheelchair is equipped with positioning belts: these should never be used as safety belts.
- Care should be taken when applying the occupant restraint to position the seatbelt buckle so that the release button will not be contacted by wheelchair components during crash.

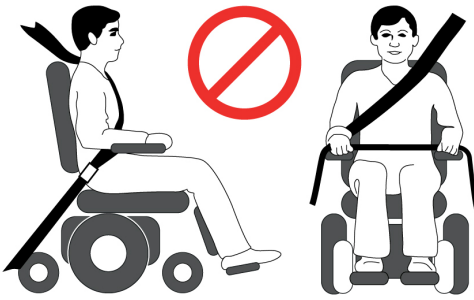
Fig. 7.2

- The upper torso restraint belt must fit over the shoulder and across the chest as illustrated (Fig. 7.3). Restraint belts must be adjusted as tightly as possible consistent with user comfort.
- Restraint belt webbing must not be twisted when in use.
- Care should be taken when applying the occupant restraint to position the seatbelt buckle so that the release button will not be contacted by wheelchair components during a crash.

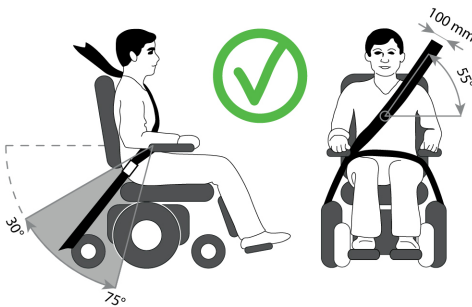
Fig. 7.3

Positioning the occupant restraint

DANGER!

- Belt restraints should make full contact with the shoulder, chest and pelvis and pelvic belts should be positioned low on the pelvis near the thigh-abdominal junction (meeting the requirements specified in ISO 7176-19:2008).
- The pelvic restraint belt must be worn low across the front of the pelvis so that the angle of the pelvic belt is within the preferred zone of 30° to 75° to the horizontal.
- A steeper (greater) angle within the preferred zone is desirable i.e. closer to, but never exceeding 75° degrees (Fig. 7.2).

8 Maintenance of the wheelchair

For information concerning specific settings, maintenance or repair work, please contact your dealer (dealer contact details on the back cover). Make sure you always mention the model, year of manufacture and identification number. This information is provided on the identification plate of the product (fig. 2.2).

We recommend having the wheelchair serviced by your dealer once a year, or in case of intensive use, every six months.

<p>Daily</p> <ul style="list-style-type: none"> • Charge batteries after each use (hst. 8.2)
<p>Weekly</p> <ul style="list-style-type: none"> • Check the tyre pressure and inflate the tyres if necessary (hst. 8.3)
<p>Monthly</p> <ul style="list-style-type: none"> • Clean the wheelchair (hst. 8.1)
<p>Annually</p> <ul style="list-style-type: none"> • Dealer check: including: • Inspection of the wheels: (tyre pressure, wear and tear). • Inspection of all the bolt and nut joints. • Inspection of all electronical parts en components. • Inspection of the motors and the carbon brushes). • Inspection of damages to the product that can lead to further damage of the product or injuries of the user when not repaired.

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8.1 Cleaning the wheelchair

To clean the seating system, we refer to the user manual for the seating system

 **XL SEATING.**

- Ensure that the controller is switched off during cleaning. If the joystick is accidentally touched, the wheelchair can move and the electrical options used accidentally.
- Be careful with water in view of the electronic system.

Removing dry dirt

First, wipe the dirty parts with a wet sponge. Use preferably clean water or soft soapy water. Wipe the parts dry with a soft dry cloth.

- Never use abrasive or aggressive cleaning agents. They can scratch the wheelchair.
- Do not use organic solvents such as thinner, benzene or white spirit.
- Never use a high pressure cleaner for cleaning the wheelchair.

8.2 Batteries

The wheelchair uses 'dry' gel batteries. These batteries are completely closed and maintenance free.

- The use of 'wet' batteries is not permitted.
- In winter, batteries have a reduced capacity. During a period of light frost, the capacity is roughly 75% of the normal capacity. At temperatures below -5°C this will be roughly 50%. This will reduce your range of action.

Charging the batteries

Check the manual of the battery charger to see if it's suitable for the type of used batteries. Check chapter 10 for the technical specifications.

- First, switch off the wheelchair's controller.
- Put the charge plug of the battery charger in the charge connector of the controller  **CONTROLLER**
- Activate the battery charger. This depends on the type of battery charger. Consult the user manual of your battery charger.

- **It is advised to mount a smoke detector in the charging area**
- **Disconnect batteries in case of longer storage without usage.**

If the wheelchair is not being used, we recommend charging the batteries via the battery charger. For normal use, the batteries should be charged every night. The battery charger's display will indicate when the batteries have been fully charged. Depending on how depleted the batteries are, it may take up to 12 hours before the batteries are once again fully charged.

NB: It takes about 15 full charging cycles before the batteries reach their full capacity.

Maintaining the batteries

Ensure that the batteries are always fully charged. Not using the batteries for a long period of time can damage them. Do not use the wheelchair if the batteries are almost depleted and never completely deplete the batteries. This can seriously damage the batteries and you may run the risk of coming to an unintended standstill.

Replacing the batteries

If the capacity of the batteries is continually decreasing so that the wheelchair can only be used for short trips, this will mean that the batteries are the end of their lifespan. The batteries will then need to be replaced. Please contact your dealer for assistance.

- **Batteries contain acids. Damaged batteries can cause severe danger to your health. Follow the instructions on the batteries at all times.**

8.3 Tyres

To ensure that your wheelchair functions properly, it is essential that the tyres are kept at the correct pressure. The maximum pressure can be found on the tyre.

Drive wheels OUTDOOR: 14" 3.00-8	Castor wheels OUTDOOR: 9" 2.80/2.50-4
Max. 3.5 bar	Max. 3.5 bar

Tyres that are too soft will have a negative effect on the wheelchair's performance. It will also cost more energy to move the wheelchair forward and this will also cause the batteries to be depleted faster. Furthermore, wear on the tyres when driving with soft tyres is unnecessarily high.

- **Never exceed the maximum tyre pressure.**

Tyre repair of drive wheels

- **Tyres should only be replaced by a qualified specialist. For an extensive description of tyre repairs we would like to refer you to the service manual available to qualified specialists.**
- **Before tyres are repaired they must first be completely deflated.**

8.4 Discarding the wheelchair

The lifespan of a wheelchair is influenced by the extent to which it is maintained. To be able to take maximum advantage of the lifespan of your wheelchair we recommend regular maintenance (see chapter maintenance).

The environment

If your wheelchair has come to be superfluous or needs to be replaced, it can usually be taken back by your dealer in consultation. If this is not possible, please ask your local authority about the possibilities of recycling or environmentally friendly disposal of the materials.

In the production of a wheelchair, many different plastics and materials are incorporated. Furthermore, the wheelchair is comprised of various electronic components that should be disposed of as electronic waste. The batteries should be put in the chemical waste bin. No environmental contribution for eventual recycling applies to wheelchairs.

9 Warranty

9.1 Definitions of terms

Definitions of terms used in this warranty:

- After sales service part: Part purchased after the initial product that is durable and may be subjected to natural wear and tear or natural contamination during normal operation within the lifetime of the product.;
- Consumable part: Part that is subjected to natural wear and tear or natural contamination during normal operation within the lifetime of the product (section 9 of Sunrise Medical HCM B.V.'s general terms and conditions of sale);
- Client: Those who purchase the product directly from Sunrise Medical;
- Corrective action: Repair, replace or refund of the product;
- Dealer: Those who re-sell the product to the User;
- Defect: Any circumstance due to which the product is not sound or fit to use, caused by a lack of quality of the material used to manufacture the product as well as the quality of the manufacturing process;
- Option: An accessory delivered with the initial product by Sunrise Medical to extend the standard product model;
- Product: Product that is delivered according to brochure or contract (e.g. wheelchair, scooter, battery-charger etc.);
- Part: Part of product that can be exchanged or replaced. This can be an option, accessory, service part or consumable part;
- Returns: Product or part that needs to be returned;
- RMA-process user: Process to return goods, contact your dealer;
- RMA-process client: Process to return goods, contact Sunrise Medical's Customer Service;
- User: Those who use the product;
- Warranty: The rights and obligations set forth in this document;
- Warranty period: The period of time during which the warranty is valid;

Notwithstanding the rights and obligations of Sunrise Medical, Client and User set forth in Sunrise Medical's general terms and conditions of sale, the rights of the Client and/or User towards Sunrise Medical in case of defects are limited to the provisions set forth in this warranty. For the duration of the warranty period Sunrise Medical guarantees that the product is without defects.

In case of any defects the User is required –within two weeks after discovery of the defect- to contact the dealer. He has to complete a return form and return the product or part via the RMA-process. Sunrise Medical will, at its sole discretion, take the corrective action it seems fit under the given circumstances within a reasonable period of time (depends on nature of claim) from receipt of the completed return form. The warranty period will not be extended after a corrective action.

9.2 Warranty period table

Power wheelchair

Description	Warranty period	Examples include, but are not limited to the parts mentioned below
Frame	2 years	Weldment/frame
Drive system*	1 year	Transaxle, motor, motor brake
Electronics*	1 year	Controller, controlling mechanism, wiring harness, electronic components
After sales service parts	New: 1 year after invoice Repaired: 90 days after invoice	Brakes
Consumable parts	40 days after invoice	Carbon brushes, etc.
Options/ Accessories	2 years	Mirror, mudguards etc. Delivered with the initial product

*also in case of after sales service part delivery

Seating

Description	Warranty period	Examples include, but are not limited to the parts mentioned below
Frame	2 years	Weldment/frame
Electronics	1 year	Electronic components
After sales service parts	New: 1 year after invoice Repaired: 90 days after invoice	Metal parts
Consumable parts	40 days after invoice	Upholstery etc.
Options/ Accessories	2 years	Lap strap, bag brackets etc. Delivered with the initial product

Sunrise Medical will only accept shipment costs and corrective costs related to warranty on equipment during the warranty period.

This warranty will void in case of:

- The product and/or its parts being modified;
- Changes in cosmetic appearance by use;
- Failure to observe the instructions for use and maintenance, use other than normal use, wear and tear, negligence, collateral damage by neglect of earlier symptoms, overloading, third-party accidents, non-original parts used and defects not caused by the product;
- Circumstances beyond our control (flood, fire, etc.).

This warranty does not cover:

- Tyres and inner tubes
- Batteries (covered by the battery manufacturer's warranty).

Clients and/or Users have legal (statutory) rights under applicable national laws relating to the sale of consumer products. This warranty does not affect statutory rights you may have nor those rights that cannot be excluded or limited, nor rights against the entity from whom the product was purchased. Clients may assert any rights they have at their sole discretion.

10 Technical specifications

As the manufacturer, SUNRISE MEDICAL, declares that the wheelchairs conform to the Medical Device Regulation (2017/745). The product meets the requirements and standards below. These are checked by independent institutions.

Standard	Definition/description	Weight test dummy
EU guideline 2017/745 EEC	Applicable as mentioned in Appendix 1	N/A
NEN-EN 12182 (1999)	The requirements from EN12182 (1999) as mentioned in NEN-EN 12184 (2009): Technical aids for the handicapped - General requirements and test methods October 1997	250 kg
NEN-EN 12184 (2009) Class B	Electric wheelchairs, scooters and accompanying battery chargers - Requirements and test methods April 2009	250 kg
ISO 7176-19	The Luca XL meets the crash test requirements as described in ISO 7176-19	104 kg

Batteries	Max.	Unit
Maximum dimensions batteries	329x173x210 (L x W x H)	mm
Battery capacity	78 / 98	Ah
Battery charger	Max.	Unit
Maximum permissible charging voltage	24	V
Maximum charging current	12	Arms.
Connector type	📖 CONTROLLER	
Insulation	Class 2 double insulated	

10.1 Electric wiring diagram

The electric wiring diagram depends on the type of controller used. Relevant information can be found in the user manual of the controller 📖 CONTROLLER.

10.2 Technical specifications Luca XL

Model:	Luca XL		
Type:	MWD		
Class:	B		
Description	Min.	Max.	Unit
Total length including powered legrests Mid Wheel Drive	>1300		mm
Total width	71		cm
Total weight without batteries:	110		kg
Batteries 78Ah GEL (C20) (set of two)	+ 48.0		kg
Batteries 98Ah GEL (C20) (set of two)	+ 61.8		kg
Transport weight of the heaviest part	110		kg
Maximum safe slope Mid Wheel Drive	6° (10.5)		° (%)
Static stability Downwards / Upwards / Sideways	15 / 15 / 15		°
Range of action* Batteries 78Ah (C20)	25	30	km
Climbing capacity for obstacles Mid Wheel Drive		60	mm
Max. speed forwards**	6		km/h
Turning radius (ISO 7176-5) Rear Wheel Drive	>80		cm

* The following aspects have a negative influence on the range of action: Obstacles, rugged terrain, driving on slopes, exposure to temperatures below the freezing point and frequent use of electronic adjustment options.

10.3 Technical specifications XL seating system

Seat angle adjustment Mechanical	0° / 3° / 6°		°
Electrical	0° - 25°		°
Effective seat depth	44 - 59		cm
Effective seat width	54 - 69		cm
Backrest angle	90 //// 124°		°
Backrest height (cushion height)	50 / 57		cm
Lower leg length	40	57	cm
Seat height (floor - top seat frame)	44 - 46.5 - 49		cm

Note:**General user advice.**

Not following these instructions may result in physical injury, damage to the product or damage to the environment!

Notice to the user and/or patient: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

B4Me special adaptations

Sunrise Medical strongly recommends that in order to ensure that your B4Me product operates, and performs as intended by the manufacturer; all the user information supplied with your B4Me product is read and understood, before the product is first used.

Sunrise Medical also recommends that the user information is not discarded after reading it, but it is kept safely stored for future reference.

Medical Device Combinations

It may be possible to combine this Medical device with one or more other Medical Device or other product. Information on which combinations are possible can be found at www.SunriseMedical.co.uk. All combinations listed have been validated to meet the General Safety and Performance Requirements, Annex I Nr. 14.1 of the Medical Device Regulation 2017/745.

Guidance on the combination, such as mounting, can be found at www.SunriseMedical.co.uk

Indications

The varieties of fitting variants, as well as the modular design, mean that it can be used by those who cannot walk or have limited mobility because of:

- Paralysis
- Loss of extremity (leg amputation)
- Extremity defect deformity
- Joint contractures/joint injuries
- Illnesses such as heart and circulation deficiencies, disturbance of equilibrium or cachexia as well as for elderly people who still have strength in the upper body.

Contraindications

The wheelchair shall not be used in case of:

- Perception disorder
- Imbalance
- Seating disability

	Sunrise Medical GmbH Kahlbachring 2-4 D-69254 Malsch Germany	 2021-04-14
Type: Luca XL	POWER WHEELCHAIR	SN 24212345678-999
	ISO7176-19:2008	
 250 kg	 max 422 kg	 max 6°
	 max 6 km/h	 max 66 kg 100 kg 256 kg
		 

Luca XL	Product Name/SKU Number.
	Maximum safe slope with anti-tip tubes fitted, Depends on wheelchair setting, posture and physical capabilities of the user.
	Maximum user weight.
	Load Maximum.
	Maximum speed.
	Maximum axle loading.
	CE Mark.
	User's Guide.
	Indicates electrical / electronic equipment must be disposed of in accordance with the WEEE regulation.
 XXXX-XX-XX	Date of manufacture.
	Serial number.
	This symbol means Medical Device.
	Manufacturer's address.
ISO 7176-19:2008	Crash tested according to ISO 7176-19:2008.

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Sunrise Medical B.V.
Sunrise Medical S.A.S