# VERMEIREN

## Carpo 4 limited

INSTRUCTION MANUAL  
MODE D'EMPLOI  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
ISTRUZIONI PER L'USO  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI

# Carpo 4:

**“Limited edition”**

The Carpo 4 Limited edition scooter is foreseen with a LCD display.

## 1. Technical Specifications

Description	Dimensions
Length	1480 mm
Width	730 mm
Height	1350 mm
Ground clearance	150 mm
Batteries	2 x 12V - 75 Ah
Driving range	45 km*
Speed	17 km/h**
User weights	150 kg
Color	Brown

\* Typical range, calculated under best conditions in accordance with the European Standards. These driving range will be reduced if the wheelchair is used frequently on slopes, rough ground or to climb curbs.

\*\* Speed and action radius can deviate (ca. 15%).

## 3. Safety rules

⚠ **Recharge the new batteries before you use the scooter, to maximize the batteries efficiency and service life.**


## 4. Components

\* Control panel

1. Battery level indicator / LCD display
2. Speed control (slow)
3. Speed control (fast)
4. Speed switch
5. Horn
6. Set: change mode
7. Lights (front and rear)
8. Indicator (left)
9. Indicator (right)
10. Emergency lights

\* LCD display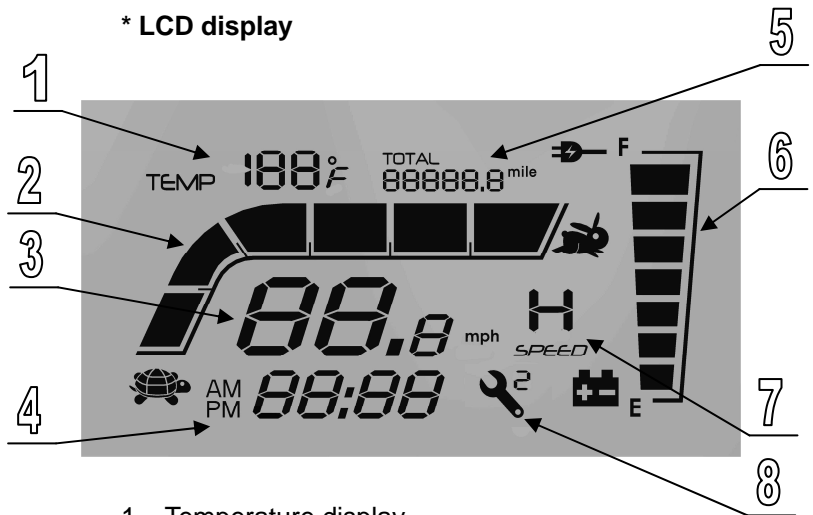

1. Temperature display
2. Speed bar
3. Speed display
4. Time display
5. Mileage or distance display
6. Battery indicator
7. High / Slow speed mode display
8. Controller fault code


## 5. Operation

The time setting is linked to the battery system, once you change or connect to new batteries, you must reset the time setting.

The system have a mileage memory for driving distance. You do not have to reset the Mileage display after changing the batteries.

### Setting:

Press **SET** to change mode, to modify time and display.

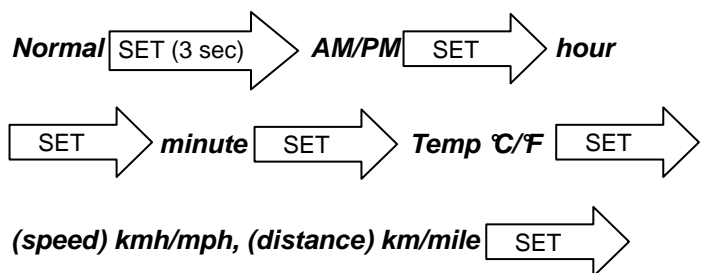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### **Vermeiren Suisse S.A.**
### **Vermeiren Nederland B.V.**