THUASNE

zorghulpmiddelen

www.max-vitaal.nl

Genu Dynastab®

fr	Genouillère ligamentaire articulée.....	6
en	Hinged ligament knee brace.....	7
de	Knieführungsorthese.....	9
nl	Scharnierende ligamentaire kniebrace.....	11
it	Ginocchiera per legamenti articolata.....	13
es	Rodillera ligamentaria articulada.....	15
pt	Joelheira ligamentar articulada.....	17
da	Artikuleret knæbind til ledbånd.....	19
fi	Nivelletty polven nivelsidetuki.....	21
sv	Ledat knäskydd för ledband.....	23
el	Επιγονατίδα συνδέσμων αρθρωτή.....	25
cs	Kolenní vazová ortéza s kloubem.....	27
pl	Przegubowa orteza stawu kolanowego.....	29
lv	Ceļa locītavas saišu ortoze ar eņģēm.....	31
lt	Lankstus raiščių antkelis.....	32
et	Liigenduvate küljetugevdustega põlvetugi.....	34
sl	Pregibna opornica za koleno.....	36
sk	Ortéza na väzy kolenného kĺbu.....	38
hu	Ízületes térdortézis a térdszalagok rögzítésére.....	40
bg	Съчленена на коленка за връзките на коляното.....	42
ro	Orteză de genunchi ligamentară cu articulație.....	44
ru	Лигаментарный ортез для коленного сустава.....	46
hr	Zglobna ortoza za ligamente koljena.....	48
zh	关节韧带护膝.....	50
ar	داعم مفصلي للركبة وأربطتها.....	53

	1	32 - 34 cm
	2	35 - 37 cm
	3	38 - 41 cm
	4	42 - 45 cm
	5	46 - 49 cm
	6	50 - 53 cm

fr	Circonférence du genou
en	Knee circumference
de	Knieumfang
nl	Omvang van de knie
it	Circonferenza del ginocchio
es	Circunferencia de la rodilla
pt	Circunferência do joelho
da	Omkreds af knæ
fi	Polven ympärys
sv	Omkrets runt knä
el	Περίμετρος του γόνατος
cs	Obvod kolena
pl	Obwód kolana
lv	Celgala apkārtmērs
lt	Kelio apimtis
et	Põlve ümbermõõt
sl	Obseg kolena
sk	Obvod kolena
hu	Térd körfogata
bg	Обиколка на коляното
ro	Circumferința genunchiului
ru	Диаметр окружности колена
hr	Opseg koljena
zh	膝关节周长
ar	محيط الركبة

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HINGED LIGAMENT KNEE BRACE

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Bilateral product.

Available in 6 sizes.

The device is composed of:

- an anatomically-shaped knitting integrating a light and soft knit at the back of the knee,
- 2 rigid hinged uprights providing knee stability ❶,
- a bi-axial hinge ❷,
- 1 anterior half-straps ❸ and 2 posterior half-straps ❹ ❸ (including a suspension strap ❹),
- a patellar insert to support patella ❶.

Composition

Textile components: polyamide - polyester - polyurethane - elastane.

Rigid components: polyamide - silicone.

Properties/Mode of action

Stabilisation of knee joint ligaments provided by the rigid hinged side uprights.

An anatomically-shaped knitting for even compression.

The brace is maintained on the leg thanks to:

- the Suspension Strap ❸;
- the silicone-coated threads at the top of the knee brace.

Staggered straps to prevent tourniquet effect.

Indications

Conservative treatment of knee ligament injuries (lateral ligaments).

Joint instability/laxity.

Knee pain and/or swelling management.

Sport activities resumption.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

History of venous or lymphatic disorders.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

This product is intended for the treatment of a given condition. Its duration of use is limited to this treatment only.

For hygiene and performance reasons, do not re-use the product for another patient.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Before any sports activity, check the compatibility of the use of this medical device with your healthcare professional.

Do not wear the product in a medical imaging device.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated. To avoid damage to the knit fabric, please make sure that the self-gripping pads are always fastened to the designated self-gripping surfaces when putting it on and taking it off.

• Slip the knee brace on the leg with your foot extended, making sure the patella is properly positioned at the centre of the patellar insert.

• To make it easier to put on, use the pull tabs at the top of the brace and turn it by approx. 90° beforehand.

• First tighten and fasten the suspension strap ❹ located above the calf.

• Tighten strap ❸, the back strap over the thigh.

• Tighten strap ❸, the front strap over the thigh.

If you feel the need to tighten the straps during an activity, make sure you tighten them following the steps indicated in the fitting instructions.

Care

Remove lateral uprights before washing.

To do this:

• Remove the straps, making a note of their original positions.

• Pull on the top of the product to disengage the upper end of the lateral uprights from of their casings.

• Remove the uprights via the top opening.

Replace after washing:

• Insert the hinge via the top opening, with the domed part outwards and flexed towards the back.

• Insert the upper and lower ends of the uprights into their casings.

• Check that the loops protrude properly through the openings.

• Replace the straps, placing strap ❹ at the back of the calf, strap ❸ at the back of the thigh and strap ❸ at the front of the thigh.

Close the self-fastening tabs before washing. Machine wash at 30°C (delicate programme). If possible use a washing net. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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- 1 semi-cinghia anteriore © e 2 semi-cinghie posteriori Ⓐ Ⓑ (di cui una cinghia di sospensione Ⓐ),