THUASNE

zorghulpmiddelen

www.max-vitaal.nl

Ligaflex® Manu

fr	Attelle d'immobilisation poignet-pouce	5
en	Wrist and thumb immobilisation splint	6
de	Orthese zur Ruhigstellung von Handgelenk und Daumen	7
nl	Immobilisatiebrace voor pols en duim	9
it	Tutore d'immobilizzazione del polso - pollice	11
es	Férula de inmovilización de muñeca - pulgar	12
pt	Tala de imobilização de punho-polegar	14
da	Skinne til immobilisering af håndled-tommelfinger	15
fi	Tukeva ranne-peukalolasta	17
sv	Ortos för immobilisering av handled/tumme	18
el	Νάρθηκας ακινητοποίησης καρπού-αντίχειρα	20
cs	Ortéz a imobilizační zápěstí a palce	21
pl	Orteza nadgarstkowo-ręczno-palcowa	23
lv	Plaukstas locītavas-īkšķa imobilizācijas šina	24
lt	Riešo ir nykščio imobilizavimo įtvaras	26
et	Randme ja põidla fikseerimise lahas	27
sl	Imobilizacijska ploščica za zapestje - palec	29
sk	Imobilizačná dlaha na zápästie-palec	30
hu	Csukló-hüvelykujj rögzítő sín	31
bg	Шина за обездвижване на китка-палец	33
ro	Atelă de imobilizare a încheieturii mâinii și a degetului mare	35
ru	Шина для иммобилизации запястья - большого пальца	36
hr	Udlaga za imobilizaciju palčanog zgloba	38
zh	手腕和拇指固定夹板	39
ar	جبيرة لتثبيت الإبهام والمعصم	43

	0	10 - 13 cm
	1	13 - 15 cm
	2	15,5 - 17,5 cm
	3	18 - 20 cm
	4	20,5 - 23 cm

fr	Tour de poignet
en	Wrist circumference
de	Handgelenkumfang
nl	Polsomtrek
it	Circonferenza del polso
es	Contorno de la muñeca
pt	Perímetro do punho
da	Håndleddets omkreds
fi	Ranteen ympärysmitta
sv	Handledens omkrets
el	Περίμετρος καρπού
cs	Obvod zápěstí
pl	Obwód nadgarstka
lv	Plaukstu locītavas apkārtmērs
lt	Riešo apimtis
et	Randme ümbermõõt
sl	Obseg zapestja
sk	Obvod zápästia
hu	Csukló kerülete
bg	Обиколка на китката
ro	Circumferința încheieturii
ru	Обхват запястья
hr	Obujam ručnog zgloba
zh	腕围
ar	محيط الرسغ

fr	Immobilisation
en	Immobilisation
de	Ruhigstellung
nl	Immobilisatie
it	Immobilizzazione
es	Inmovilización
pt	Imobilização
da	Støtte
fi	Immobilisoi
sv	Immobilisering
el	Ακινητοποίηση
cs	Znehybnění
pl	Unieruchomienie
lv	Imobilizācija
lt	Imobilizavimas
et	Liikumatuks muutmine
sl	Imobilizacija
sk	Imobilizácia
hu	Rögzítés
bg	Обездвижване
ro	Imobilizare
ru	Иммобилизация
hr	Imobilizacija
zh	固定
ar	التثبيت

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WRIST AND THUMB IMMOBILISATION SPLINT

Description/Destination

This device is a wrist-thumb splint providing joint immobilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Textile components: Polyamide - Polyester - Cotton - Elastane.

Rigid components: Polyoxymethylene - Aluminium.

Properties/Mode of action

Immobilisation of the wrist and thumb via 2 rigid, malleable and anatomical palmar stays.

Aerated, foam lined fabric.

Dual strap system ensuring even compression.

Indications

Thumb sprain.

Wrist and/or thumb sprain.

Non surgical scaphoid fracture.

Non surgical wrist fracture.

Post-operative or post-traumatic wrist/thumb immobilisation.

Wrist and/or thumb tendinopathies.

De Quervain tenosynovitis.

Rheumatoid arthritis.

Lateral epicondylitis.

Conservative treatment of carpal tunnel syndrome.

Thumb/wrist osteoarthritis.

Neuro-muscular impairments (contracture, hand drop or spasticity...).

Contraindications

Do not use the product if the diagnosis has not been confirmed.
Do not apply the product in direct contact with broken skin.
Do not use in the event of known allergy to any of the components.

Precautions

Verify the product's integrity before every use.
Do not use the device if it is damaged.
Choose the appropriate size to fit the patient, referring to the size chart.
It is recommended that a health care professional supervises the first application.
It is recommended that an adult supervises the application and use of the product by a child.
Strictly comply with your healthcare professional's prescription and recommendations for use.
In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a health care professional.
For hygiene and performance reasons, do not re-use the product for another patient.
Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.
Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

If necessary, shape the stays.
Open the splint et position the thumb and the wrist inside the product.
Adjust the tightening system (wrist then thumb) to close the splint.

Care/Maintenance

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Remove stay before washing. Replace in the original location before next use. Close the self-fastening tabs before washing.
Machine washable at 40°C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.
Keep this instruction leaflet.

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©Thuasne - 2020002 (2020-11)